Erkocryl klar (clear)

23°F
-5°C
-45°C
113°F

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EN: Instructions for use

Intended use: Thermoforming sheets are indicated for the fabrication of dental and orthodontic appliances for the treatment of all patient target groups.

Field of application: Erkocryl is exclusively suitable for the fabrication of thermoformed intraoral splints and appliances such as orthodontic plates, interim dentures, denture bases, dressing or compression plates. Fabrication, see thermoforming technique brochure and videos, www.erkodent.com  

Contraindications: In case of already known allergic reaction to plastics a specific allergy to Erkocryl (PMMA) should be excluded.

Precautionary measures: Erkocryl is only recommended for the above mentioned appliances. Observe storage instructions.

Attention, sensitive to humidity: Close the aluminium bag immediately.

Attention: The use is limited to the production of dental and orthodontic appliances. Use is subject to the responsibility of the therapist. Use only on the recommendation or prescription of an authorised therapist. For single use only (thermoforming). Allergic reactions are unlikely, but cannot be excluded. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small breakages. Serious incidents in connection with Erkocryl must be reported by the user to the competent authority and to the manufacturer.

Erkocryl (approx. values): Density 1.17 g/cm³, modulus of elasticity 2400 MPa, elongation at break 22 %, water absorption 0.3 %, temperature resistance 96 °C

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