

procedure pack

$K^3 = \sqrt{\text{ENDO}}$

SybronEndo
A DIVISION OF SYBRON DENTAL SPECIALTIES



.10 taper	.08 taper	.06 taper	.06 taper	.06 taper	.06 taper
25 tip	25 tip	40 tip	35 tip	30 tip	25 tip
17 mm	17 mm	21 or 25 mm	21 or 25 mm	21 or 25 mm	21 or 25 mm

0 5 10 15 20 25 30 millimeter

INSTRUMENTATION GUIDE

1. Obtain Straight-Line Access using the LA Axxess kit.
2. Locate the canal orifices and obtain coronal patency using hand files.
3. Begin Crown-down by taking .10 Taper K3 Shaper to resistance.
4. Take .08 taper K3 Shaper to resistance.
5. Estimate apical size.
6. Begin Crown Down using a size #40 K3 File.

Clinical Tips

- **Establish a glide path with hand files. Confirm patency between all rotary files.**
- **Irrigate often, alternating between sodium hypochlorite and SmearClear (17% EDTA wetting agents).**
- **Use very light pressure. Never force instruments. When the instrument no longer advances apically, proceed to the next file.**

* The Procedure Pack will work in many cases, but all canals are different and additional sizes of K3 Files may be required. You will want to begin with a K3 File 3 sizes larger than your estimated apical diameter.

7. Establish working length. Establish this with your Elements Diagnostic Unit. A radiograph may be taken to assist in working length determination.
8. Complete Crown Down preparation with a #35 K3 File, #30 K3 File, #25 K3 File, by taking each instrument to resistance at 300-350 RPM in an electric torque control motor, using each instrument for no longer than 5-7 seconds.