Master all your challenges

initial™

Spectrum Stains

- EN Technical Manual
- DE Technisches Handbuch
- FR Manuel Technique
- IT Manuale tecnico

APRIL 2018

E	DE	FR	IT	
Table of Contents	Inhaltsverzeichnis	Table des matières	Indice	Page
1. Intended Use & Set Composition	1. Bestimmungsgemäße Verwendung und Set-Zusammensetzung	1. Indications & Composition des coffrets	1. Destinazione d'uso e composizione del set	4 - 5
2. Description of GC Initial Spectrum Stains	2. Beschreibung der GC Initial Spectrum Stains	2. Description des GC Initial Spectrum Stains	2. Descrizione di GC Initial Spectrum Stains	6
3. The Glaze & Glaze Liquids	3. Glasurflüssigkeit und Glasurpaste	3. Poudre Glaze & Liquides Glaze	3. La polvere e i liquidi per glasura	7
4. GC Initial Spectrum Stain Application	4. Anwendung der GC Initial Spectrum Stains	4. Application des GC Initial Spectrum Stains	4. Applicazione di GC Initial Spectrum Stain	
External Characterization	Äußere Charakterisierung	Caractérisation externe	Caratterizzazione esterna	8 - 11
Internal Characterization	Innere Charakterisierung	Caractérisation interne	Caratterizzazione interna	12
Combined Characterization	Kombinierte Charakterisierung	Caractérisation combinée	Caratterizzazione combinata	14 - 15
5. Firing Chart	5. Brenntabelle	5. Tableau de cuisson	5. Tabella dei parametri di cottura	16
6. Physical Properties & Shelf Life	6. Physikalische Eigenschaften und Haltbarkeit	6. Propriétés physiques & Péremption	6. Proprietà fisiche e durata utile del prodotto	17
7. FAQ	7. Häufig gestellte Fragen	7. FAQ	7. Domande frequenti	18 - 22
8. Related Products	8. Verwandte Produkte	8. Produits associés	8. Prodotti collegati	42

1

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EN One for All

GC Initial Spectrum Stains are universal stains & glaze for the individualized staining and characterization of ceramic materials.

- **A wide spectrum of use**

The stains and glaze are developed for the external staining and glazing of tooth coloured ceramic materials and internal individualization of layering ceramics.

Compatible to a wide CTE-range, the GC Initial Spectrum Stains are perfectly adapted to fit on nearly all types of dental ceramics.

In particular they are adapted to the aesthetical colour concept of the GC Initial ceramic portfolio.

- **A wide spectrum of colours**

GC Initial Spectrum Stains are powder stains available in a variety of 16 colours empowering the aesthetics of your restorations.

- **Adaptable consistency of the stains & glaze**

By using either the Glaze or the Glaze Paste Liquid the stains & glaze can be adapted to the preferred consistency of the user.

Regardless the way of application a very fine end result will be obtained maintaining the surface texture and assuring an optimal gloss level.

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GC Initial Spectrum Stain Set – Set Composition

- 16x GC Initial Spectrum Stain SPS1 – SPS16, 3g
- 1x GC Initial Spectrum Glaze Powder GL, 10g
- 1x GC Initial Spectrum Glaze Liquid, 25ml
- 1x GC Initial Spectrum Glaze Paste Liquid, 8ml
- 1x GC Initial Spectrum Stain Shade Guide

*We advise you to thoroughly check the Q&A for further clarification on application details.

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- | | |
|--|---|
|  Initial Spectrum Stain SPS-1, 3g - ivory white |  Initial Spectrum Stain SPS-9, 3g - brown |
|  Initial Spectrum Stain SPS-2, 3g - melon yellow |  Initial Spectrum Stain SPS-10, 3g - dark walnut |
|  Initial Spectrum Stain SPS-3, 3g - signal yellow |  Initial Spectrum Stain SPS-11, 3g - coral |
|  Initial Spectrum Stain SPS-4, 3g - light terracotta |  Initial Spectrum Stain SPS-12, 3g - denim blue |
|  Initial Spectrum Stain SPS-5, 3g - savannah gold |  Initial Spectrum Stain SPS-13, 3g - twilight |
|  Initial Spectrum Stain SPS-6, 3g - caramel |  Initial Spectrum Stain SPS-14, 3g - deep orange |
|  Initial Spectrum Stain SPS-7, 3g - camel |  Initial Spectrum Stain SPS-15, 3g - sunset |
|  Initial Spectrum Stain SPS-8, 3g - olive |  Initial Spectrum Stain SPS-16, 3g - midnight |

- EN The Glaze & Glaze Liquids
- DE Glasurflüssigkeit und Glasurpaste
- FR Poudre & Liquides Glaze
- IT La polvere e i liquidi per glasura

• EN **GC Initial Spectrum Glaze Powder GL, 10g**

Fine Glaze powder to be mixed with the Glaze or Glaze Paste Liquid.

The Glaze Powder can be adapted to the preferred consistency of the user by using the Glaze or the Glaze Paste Liquid.

GC Initial Spectrum Glaze Liquid, 25ml

The standard low viscosity type of mixing liquid allowing a fine application.

GC Initial Spectrum Glaze Paste Liquid, 8ml

The high viscosity type of mixing liquid allowing a gel-type application.

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GC Initial Spectrum Liquide Glaze, 25ml

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• IT **GC Initial Spectrum Glaze Powder GL, 10g**

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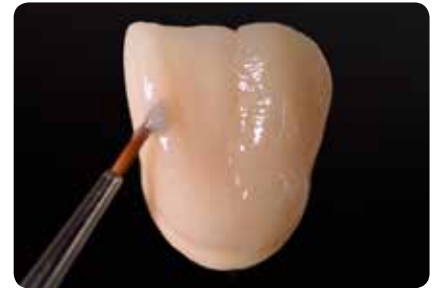
EN Initial situation

Ceramic crown with individual micro-texture, ready to be stained & glazed. *Note: before staining and glazing, the ceramic needs to be sand-blasted with 50 microns Alu-Oxid at 1,5 bar pressure and steam cleaned or cleaned by ultra-sonic bath.*

By using either the Glaze or the Glaze Paste Liquid the stains can be adapted to the preferred consistency.

Left: "Gel-type" mixture with Glaze Paste Liquid.

Right: "Fine" mixture with Glaze Liquid.

Coat the entire surface of the restoration with the Spectrum Glaze in the preferred consistency.

Left part of the crown: application of the Glaze using the Glaze Paste Liquid.

Right part of the crown: Glaze applied using the Glaze and Glaze Liquid.

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EN **Initial LiSi Press Crown**
 As connection firing Lustre Pastes NF have been used on the entire surface.
 For extra internal individualization the Initial Spectrum Stains are applied and fired.

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EN **Layering of the Initial LiSi Press Crown with the Initial LiSi veneering ceramic.**
 Initial Spectrum Stains are used for internal individualization during the layering process.

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EN Please consult the firing parameters of the dedicated layering materials.

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EN External staining of the Initial Lisi Press Crown to mimic nature analogue ageing of the tooth.

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EN Final result after internal and external staining.

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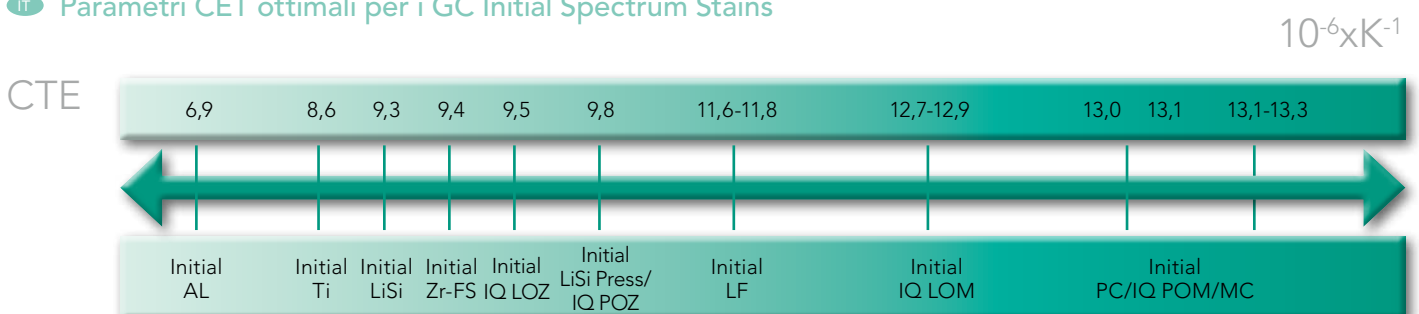
EN *All values mentioned in the physical properties overview meet the performance criteria as defined in EN ISO 6872:2008 Dentistry – Ceramic Materials ISO 6872:2008).

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EN Important to know:

- GC Initial LRF Block
It is advised to use the GC Initial LRF Glaze Paste as dedicated glazing material.
The GC Initial Spectrum Stains can be added in limited amounts to the GC Initial LRF Glaze Paste for individualisation.
- Layering materials can be individually toned using the Spectrum Stains.
When mixed into Dentin, Enamel, Transpa powders it is advised to use small amounts.
To assure the right mixing ratio we advise to create a small firing sample.
- When used during build-up on an unfired ceramic small amounts can be added.
- When mixing layering material and stains use the recommended modelling liquid not the Initial Spectrum Stain Liquid.
- Always consult the firing parameters of the dedicated layering materials.

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EN Q2. Are the GC Initial Spectrum Stains & Glaze fluorescent?

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EN Similar to natural teeth the GC Initial Spectrum Stains offer a certain level of fluorescence.

The Glaze Powder however is a non-fluorescent glazing material.

It protects and glazes your surface.

Fluorescence is given by the underlying ceramic powders / core material.

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EN Q3. Can the GC Initial Spectrum Stains be mixed with the GC Initial IQ One Body Lustre Pastes NF?

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EN Yes they can be mixed with moderation to intensify the Lustre Pastes NF. Mix the required amount of stain powder with the LP NF Dilution Liquid and mix it with the required LP NF or directly mix the stain powders with e.g. the Lustre Paste Neutral (L-N).

Initial Spectrum Stains and Lustre Pastes NF are compatible products.

Initial Lustre Pastes NF are ready-to-use 3-dimensional dynamic shades and are based on a special mix of fine ceramic particles.

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EN Q4. Are there any contraindications for use of the GC Initial Spectrum Stains?

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EN Q5. I do not have a shiny surface after firing the Glaze, what might be the cause?

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EN Yes, you can do further firings, keeping the same firing temperatures.

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PL Opis GC Initial Spectrum Stains

- Initial Spectrum Stain SPS-1, 3g - ivory white
- Initial Spectrum Stain SPS-2, 3g - melon yellow
- Initial Spectrum Stain SPS-3, 3g - signal yellow
- Initial Spectrum Stain SPS-4, 3g - light terracotta
- Initial Spectrum Stain SPS-5, 3g - savannah gold
- Initial Spectrum Stain SPS-6, 3g - caramel
- Initial Spectrum Stain SPS-7, 3g - camel
- Initial Spectrum Stain SPS-8, 3g - olive
- Initial Spectrum Stain SPS-9, 3g - brown
- Initial Spectrum Stain SPS-10, 3g - dark walnut
- Initial Spectrum Stain SPS-11, 3g - coral
- Initial Spectrum Stain SPS-12, 3g - denim blue
- Initial Spectrum Stain SPS-13, 3g - twilight
- Initial Spectrum Stain SPS-14, 3g - deep orange
- Initial Spectrum Stain SPS-15, 3g - sunset
- Initial Spectrum Stain SPS-16, 3g - midnight

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PL GC Initial Spectrum Glaze Powder GL, 10 g

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GC Initial Spectrum Glaze Liquid, 25 ml

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GC Initial Spectrum Glaze Paste Liquid, 8 ml

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GC Initial LiSi Press

GC Initial MC, Zr-FS, LiSi

A series of horizontal dotted lines spanning the width of the page, providing a guide for handwriting practice.

 GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240
Interleuvenlaan 33, B-3001 Leuven
Tel. +32.16.74.10.00, Fax. +32.16.40.02.14
info.gce@gc.dental, www.gceurope.com

GC AMERICA INC.
3737 West 127th Street, USA-Alsip, ILL. 60803
Tel. +1.708.597.0900, Fax. +1.708.3771.5103
sales@gcamerica.com, www.gcamerica.com

GC GERMANY GmbH
Seifgrundstrasse 2, D-61348 Bad Homburg
Tel. +49.61.72.99.59.60, Fax. +49.61.72.99.59.66.6
info.germany@gc.dental, www.germany.gceurope.com

GC ITALIA S.r.l.
Via Calabria 1, I-20098 San Giuliano Milanese
Tel. +39.02.98.28.20.68, Fax. +39.02.98.28.21.00
info.italy@gc.dental, www.italy.gceurope.com

GC UNITED KINGDOM Ltd.
16-23 Coopers Court, Coopers Court, Newport Pagnell, UK-Bucks.
MK16 8JS
Tel. +44.1908.218.999, Fax. +44.1908.218.900
info.uk@gc.dental, www.uk.gceurope.com

GC FRANCE s.a.s.
8 rue Benjamin Franklin, F-94370 Sucy en Brie Cedex
Tel. +33.1.49.80.37.91, Fax. +33.1.45.76.32.68
info.france@gc.dental, www.france.gceurope.com

GC IBÉRICA - Dental Products, S.L.
Edificio Codesa 2
Playa de las Américas, 2, 1º, Of. 4, ES-28290 Las Rozas, Madrid
Tel. +34.916.364.340, Fax. +34.916.364.341
comercial.spain@gc.dental, www.spain.gceurope.com

GC AUSTRIA GmbH
Tallak 124, A-8103 Rein bei Graz
Tel. +43.312.45.40.20, Fax. +43.312.45.40.20.40
info.austria@gc.dental, www.austria.gceurope.com

GC BENELUX B.V.
Edisonbaan 12, NL-3439 MN Nieuwegein
Tel. +31.30.630.85.00, Fax. +31.30.605.59.86
info.benelux@gc.dental, www.benelux.gceurope.com

GC EUROPE N.V. - East European Office
Siget 19b, HR-10020 Zagreb
Tel. +385.1.46.78.474, Fax. +385.1.46.78.473
info.eeo@gc.dental, www.eeo.gceurope.com

GC NORDIC AB - Finnish Branch
Bertel Jungin aukio 5 (6. kerros), FIN-02600 Espoo
Tel. +358 40 9000 757
info.finland@gc.dental, www.finland.gceurope.com

GC NORDIC DANISH BRANCH
Scandinavian Trade Building
Gydevang 39-41, DK-3450 Allerød
info.denmark@gc.dental, http://nordic.gceurope.com

GC NORDIC AB
Strandvägen 54, S-193 30 Sigtuna
Tel:+46 768 544 350
info.nordic@gc.dental, http://nordic.gceurope.com

GC AUSTRIA GmbH - Swiss Office
Bergstrasse 31, CH-8890 Flums
Tel. +41.81.734.02.70, Fax. +41.81.734.02.71
info.switzerland@gc.dental, www.switzerland.gceurope.com

GC AUSTRALASIA DENTAL PTY LTD
1753 Botany Rd, Banksmeadow
Sydney, NSW 2019, Australia
Tel: +61.2.9301.8200, Fax: +61.2.9316.4196
www.gcaustralasia.com