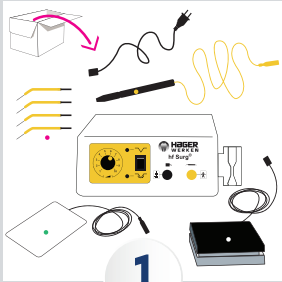
hf SURG®

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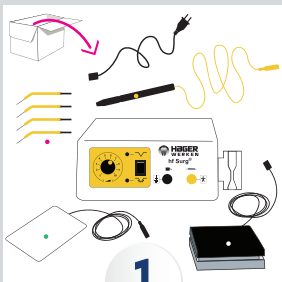
hf SURG®

QUICK GUIDE


FOR ALL OTHER IMPORTANT DETAILS, PLEASE REFER TO THE INSTRUCTIONS FOR USE.

1

SCOPE OF DELIVERY:

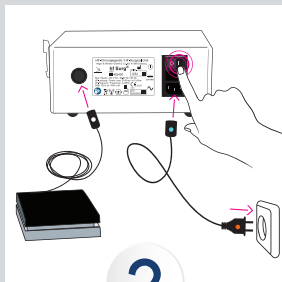
- Device
- Foot pedal
- Handpiece with cable
- 4 electrodes
- Neutral electrode

2

INSERTING THE CABLES

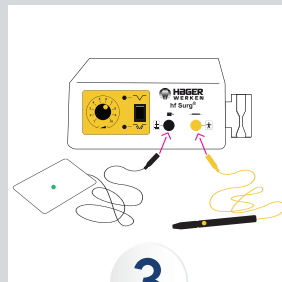
- Insert the device plug into the corresponding socket.
- Insert the power plug into the socket.
- Connect the foot pedal.
- Operate the main switch.

3

INSERTING THE CABLES

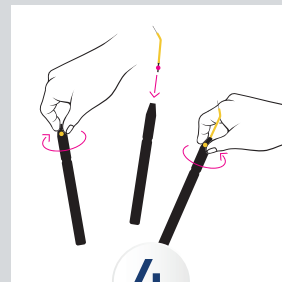
- Connect the handpiece to the yellow socket.
- Connect the neutral electrode to the black socket.

4

INSERTING THE ELECTRODE

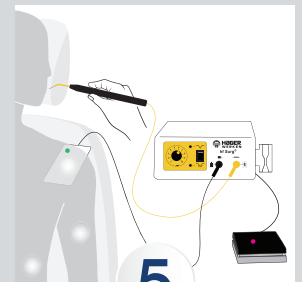
- Turn the head of the handpiece one turn to the left.
- Insert the shaft of the electrode all the way into the handpiece.
- Turn the head of the handpiece to the right and secure the electrode with it.
- Check that the working electrode is properly fixed in the handpiece. It is securely fastened when it can no longer move in the handpiece.

5

INSERTING THE CABLES

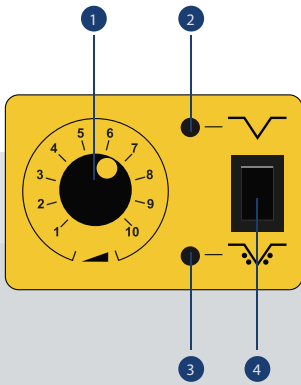- The neutral electrode must be positioned behind the right or left shoulder of the dressed patient.
- Activate the electrode by pressing the foot switch before touching the tissue.

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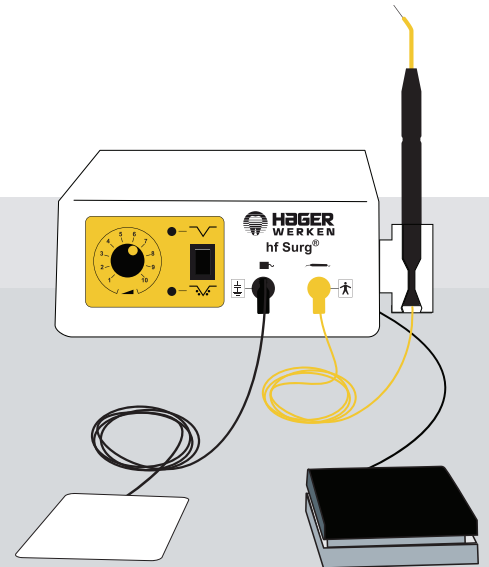
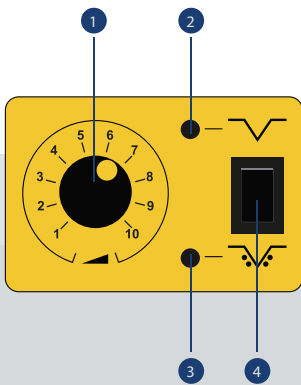
FRONT OF UNIT

1. Controller - intensity regulator
2. LED yellow light shows the selected wave type - CUTTING SHAFT
3. LED yellow light shows the selected wave type - CUTTING/COAGULATION SHAFT
4. Switch CUT or CUT/COAG

STERILISATION

Handpiece and electrodes are autoclavable at 134 °C

The handpiece can be sterilised together with the cable in the autoclave in the foil sterilisation pouch.

WARNING

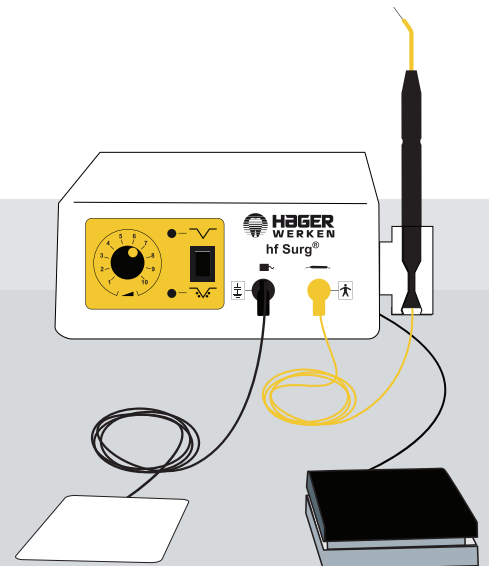
Computers, laptops and mobile phones should be kept at least 30 cm away from the hf Surg device.

The device must be connected only to a power supply with an earth wire in order to avoid the risk of an electric shock.

During operation, a minimum distance of approx. 20 cm from any walls must be observed.

For MORE INFORMATION, please refer to our HF catalogue

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HAGER & WERKEN GmbH & Co. KG